

# TEMPORARY STRAP BACKORDER NOTICE

## Your therapy can continue without interruption.

Due to extremely high clinical demand, 24-inch ReBuilder knee straps are temporarily backordered.

To ensure your treatment continues without delay, we have included the necessary number of self-adhesive electrodes in this shipment. These electrodes allow the ReBuilder therapy signal to stimulate the same neural and vascular pathways while your straps are on the way.

Your 24-inch straps will be shipped to you as soon as inventory is restored.

### NeuroVascular Electrode Placement - Temporary Configuration



Example configuration showing foot electrode placement and temporary quadriceps electrode placement.

Adhesive electrodes placed on the quadriceps stimulate the sciatic nerve before it branches into the tibial and peroneal pathways supplying the lower leg and foot.

## Why This Works

Stimulation applied above the knee engages proximal neural pathways along the sciatic distribution while activating regional musculature and vascular structures involved in circulatory and lymphatic response.


The ReBuilder waveform also stimulates smooth muscle within vascular and lymphatic vessels, supporting circulation and fluid movement throughout the lower extremity.

While the circumferential strap provides the most uniform neurovascular recruitment, quadriceps electrode placement continues to provide meaningful therapeutic benefit during the temporary strap supply interruption.

## Knee Electrode Placement - Temporary Configuration

Example configuration showing back of calf electrode placement and temporary above-knee electrode placement.

**Your 24-inch knee straps will be shipped immediately once inventory is restored.**

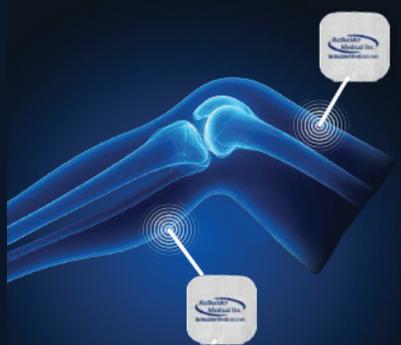

ReBuilder Medical  
**RELIEVE • RETRAIN • REBUILD**