


Severe Neuropathy

Full-Limb Therapy for Advanced Neuropathy, Edema, and Wound Care Support

Don't mask symptoms. Treat the root cause with ReBuilder®.

ReBuilder Elevated NeuroVascular Protocol™

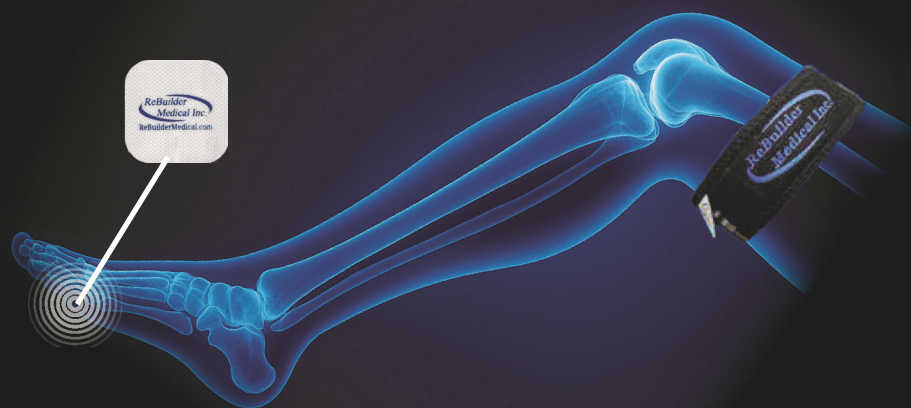

Best For:

Patients experiencing advanced or diffuse neuropathic symptoms where elevating the legs during treatment enhances blood flow, reduces inflammation, and optimizes nerve recovery outcomes.

Including:

- Severe neuropathy extending beyond the feet
- Ulcers, wounds, or post-surgical limitations
- Extreme edema

Placement & Setup

Output 1 – Feet:

Place one adhesive electrode on the ball of each foot, with the wire from the electrode between the first and second toe.

Output 2 – Knees:

Place a conductive knee strap 1–2 inches above each knee.



ReBuilder 2407

Treatment Parameters

Session Length: 30 minutes

Frequency: 2 times per day

Settings: Standard ReBuilder® mode; adjust intensity to patient comfort

ReBuilder is FDA-cleared for the symptomatic relief of chronic intractable pain, post-traumatic pain relief, post-surgical pain relief, and maintaining or increasing range of motion.


ReBuilder®
RELIEVE • RETRAIN • REBUILD

ReBuilder's proprietary signaling

delivers physiologically precise therapy to improve circulation and nerve function—for lasting relief

Required Equipment

ReBuilder® 2407 device
2 lead wires
2 self-adhesive electrodes
2 knee straps
ReBuilder® Electrolyte Spray (optional)

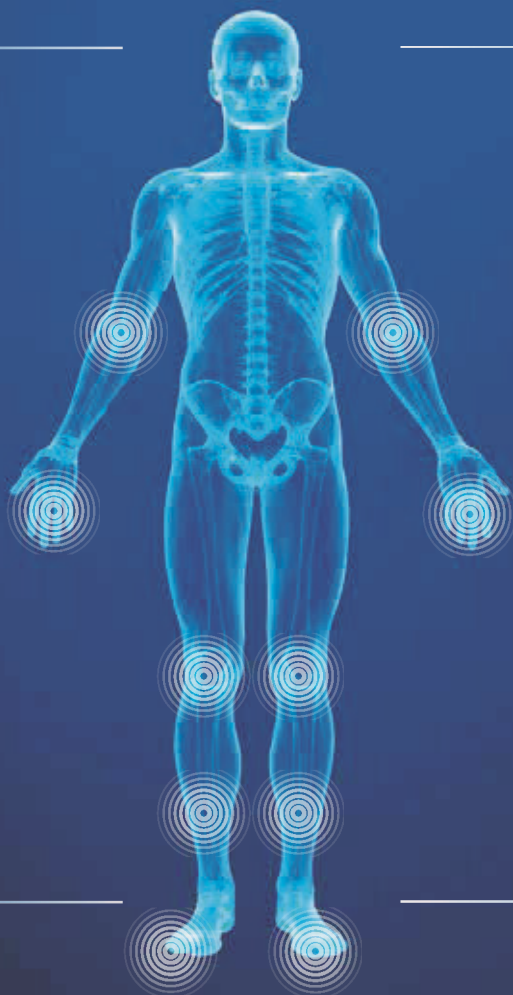

RELIEVE Pain

Enhancing microcirculation and calming damaged nerves, delivering the nutrients needed to heal

RETRAIN Nerves

The calming ReBuilder signal re-educates damaged nerves enabling proper function to be restored

REBUILD Function

Improving vascular and neural dysfunction to reduce neuropathy symptoms and restore mobility

RECLAIM Life

Supporting lasting recovery so patients can reclaim their lives from neuropathic pain



**Rx
Only**



**5 Year Device
Warranty**



**Same Day
Shipping (M-F)**



**Customer Service
Support: 8AM-6PM**

Visit **ReBuilderMedical.com** to learn more.

INDICATIONS:

ReBuilder is FDA-cleared for the symptomatic relief of chronic intractable pain, post-traumatic and post-surgical pain relief, relaxation of muscle spasms, prevention or retardation of disuse atrophy, increasing local blood circulation, muscle reeducation, immediate post-surgical stimulation of calf muscles to prevent venous thrombosis, and maintaining or increasing range of motion.

WARNINGS AND PRECAUTIONS:

Use of the ReBuilder is contraindicated for individuals who are pregnant, children under 18 years of age, with pacemakers or any other implanted electrical medical devices, unless written consent given by cardiologist or implanted electrical medical device manufacturer. Use only under direct supervision of your physician or health care provider if any of the above conditions apply. Do not use directly over an open wound. Do not use if active deep vein thrombosis is present. Do not apply to head, neck or shoulders of person with epilepsy. Follow your physician's advice for specific instructions.

ReBuilder®
RELIEVE • RETRAIN • REBUILD